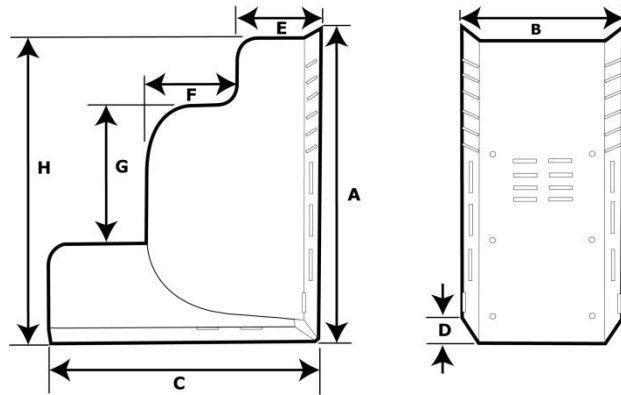
# Technische Daten / Technical information

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## Safety Notes

**To reduce risk, please ensure that the instructions have been read and fully understood before use. The assembly instructions must be strictly adhered to.**

## Application

The three piece Aluminium Frame provides the base for individually manufactured seating systems. Using the slots provided, the system width can be adjusted at any time to allow for growth. The seat to back angle is delivered in the 90° position, it can however be adjusted from 85° to 120° using the crescent shaped adjustment slots provided.

## Assembly and Fitting

When assembling the frame the two side panels should be fitted inside the middle panel. Set the seat to the width required. These three panels should be assembled using the Screw Set Art. No. E 1 2 011 (initially hand tight for assembly). The final torque setting is 17 Nm. **See "Important notice" below.** After defining the seat – back angle, bore a 6.4mm hole in the middle of the crescent shaped adjuster slot and secure this with a screw supplied in the screw set. For clients who have strong extensor spasms or similar, a second screw on each side should be fitted to provide extra strength and to prevent movement or opening of the angle. Trimming of the Aluminium Frame should be done using a Jig-Saw. The contour of the Aluminium Frame should be 2 – 3 cm smaller than the foam contour of the seating system. Remove all burring and sharp edges, both on the saw lines and the bore holes before fitting the Trim Band (Art. - No. E 1 2 012).

## Important Notice

All nuts and screws supplied may **only be used once** after tightening to the **torque setting of 17Nm**. During assembly the nuts and screws should only be lightly tightened.

All sharp edges resulting from sawing and boring should be deburred and covered with Trim Band (Art. - No. E 1 0 012), where applicable.

## Testing

These Aluminium frames have been crash tested to ISO 7176/19.

## Care and Maintenance

To clean the frame use car polish or mild soap and water. **All screws should be checked for tightness at regular intervals.**

## EU Declaration of Conformity

We, McLean REHAtechnik GmbH, declare under our sole responsibility that the products described in these operating instructions meet the basic requirements of the regulation (EU) 2017/745 (MDR).

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All rights reserved .